

DrugConfirm™

OTC Urine Drug Test Kit

drugconfirm™
instant multi-drug test kit



DrugConfirm™ is a comprehensive over-the-counter urine drug test kit, maintaining 99% reporting accuracy at home or in the workplace. 1, 4 and 12 panels are configured to detect the most common drugs of abuse. The integrated dip card yields results within minutes, and can be sent to a laboratory for confirmation testing. Urinalysis is easier than ever with DrugConfirm™, so the guesswork can be removed from drug abuse screening. The DrugConfirm™ line is shelf ready for online retail or over-the-counter distribution.

PRODUCT FEATURES:

- CLIA-Waived, FDA 510(k) Approved & CE Marked
- Featured in 1, 4 and 12 Panel Configuration Options
- Instant Results Within 5 Minutes or Less
- 99% Result Accuracy
- Made in the USA

EACH KIT CONTAINS:

- DrugConfirm™ 1, 4, or 12 Panel Drug Test Dip Card
- Urine Collection Cup and Funnel
- Labeled Vial
- Confirmation Identification Label
- Transport Pouch
- Pre-addressed Mailing Box
- Instruction Booklet

1 PANEL - DRUGCONFIRM™	4 PANEL - DRUGCONFIRM™	12 PANEL - DRUGCONFIRM™
THC	AMP, COC, OPI, THC	AMP, BAR, BZO, COC, MDMA, THC, MAMP, MTD OXY, OPI, PCP, TCA

TEST	DESCRIPTION	CUT-OFF LEVEL
AMP	AMPHETAMINE	1000 NG/ML
BAR	BARBITURATES	300 NG/ML
BZO	BENZODIAZEPINE	300 NG/ML
COC	COCAINE	300 NG/ML
MDMA	ECSTASY	500 NG/ML
THC	MARIJUANA	50 NG/ML
MTD	METHADONE	300 NG/ML
mAMP	METHAMPHETAMINE	1000 NG/ML
MOR/OPI	MORPHINE/OPIATES	300/2000 NG/ML
OXY	OXYCODONE	100 NG/ML
PCP	PHENCYCLIDINE	25 NG/ML
TCA	TRICYCLIC ANTIDEPRESSANTS	1000 NG/ML

FOR STREAMLINED, ALL-IN-ONE URINE DRUG TESTING, USE DRUGCONFIRM™

DrugConfirm™ OTC Urine Drug Test Kit is the right solution for all inclusive drug screening, with laboratory confirmation testing add-on. Multiple panel configurations paired with reliable results yield a convenient urine test at an unbeatable price. Refined drug testing at home or in the workplace is only an order away.

INSTRUCTIONS

1. When you are ready to test, collect a urine sample with the disposable cup provided with your test box.
2. Open the test pouch immediately before use. Remove test device from the pouch. Remove the cap from the device. Dip the sample end of the device into the specimen. NOTE: You must immerse the sample well into the specimen completely and keep the tip of the arrows in the device's window above the specimen surface.
3. Read test immediately. Do not compare color intensity of one test to another. Do not compare color intensity of the T line to the C line. Do not interpret results before 4 minutes or after 7 minutes that may introduce the false positive or negative result.

INTERPRETATION OF RESULTS

NEGATIVE:

A red or pink colored line next to the Control Line (C Line) and a red or pink colored line next to the abbreviated drug panel indicates a Negative result for that drug, regardless of the color intensity.




POSITIVE:

A red or pink colored line next to the Control Line (C Line) and no line next to the abbreviated drug panel indicates a Preliminary Positive result.



INVALID:

If after 10 minutes no line appears next to the Control Line (C Line), the result is invalid.

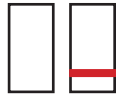

IMPORTANT INFORMATION

Do not use after the expiration date.

The test device should remain in the sealed pouch until use.

Results are preliminary and positive results do not always mean that drugs are present in the donor. Certain foods and medicines, diet plan drugs, and nutritional supplements may yield a false positive test result with this product. Similarly, if the sample is diluted or adulterated, the product may yield negative results and not detect a particular drug. Laboratory tests are needed to confirm preliminary positives.

Lab results obtained through DrugConfirm™ products are accessible through a secure Account Creation Portal. For troubleshooting, testing concerns and product inquiries, please phone our toll-free Customer Service line (800) 656-0745, or fill out an Online Feedback Form.